t:slim X2™

Compatible Insulins / Insulinas compatibles / Insulines compatibles / Zugelassene Insuline / Insuline compatibili / Compatibile insulines / Kompatible insuliner / Kompatible insuliner / Yhteensopivat insuliinit / Kompatibla insulin / Kompatibilni inzuliny / Insulinas compativeis / الأنسولين المتوافق / סוגי אינסולין תואמים

1.1 English

This information is provided as reference for pump users and caregivers who are already familiar with the use of an insulin pump and with insulin therapy in general. For more detailed information on the operation of the t:slim X2 insulin pump, please refer to its user guide.

The t:slim X2 insulin pump with Control-IQ technology is designed for use with rapid acting insulin analogs that have been tested and found to be safe for use in the pump:

- NovoLog/NovoRapid U-100 insulin
- Humalog U-100 insulin
- Lyumjev U-100 insulin (European Union, Israel, Norway, Switzerland, and United Kingdom only)
- Admelog/Insulin lispro Sanofi U-100 insulin (Finland, Germany, Italy, Netherlands, Sweden, and United Kingdom only)
- Trurapi/Insulin aspart Sanofi U-100 insulin (Finland, Germany, Italy, Netherlands, Sweden, and United Kingdom only)

NovoLog/NovoRapid and Trurapi/Insulin aspart Sanofi are compatible with the system for use up to 72 hours (3 days). Humalog, Admelog/Insulin lispro Sanofi, and Lyumjev are compatible with the system for use up to 48 hours (2 days). If you have questions about using other insulins, contact your healthcare provider. Always consult your healthcare provider and refer to the insulin labeling prior to use.

Infusion Site Care and Loading Cartridge

The infusion set must be replaced and rotated every 48 hours if using Humalog, Admelog/Insulin lispro Sanofi, or Lyumjev insulin; every 72 hours if using NovoLog/NovoRapid or Trurapi/Insulin aspart Sanofi insulin, or more often if needed.

⚠ PRECAUTION

CHANGE your infusion set every 48 hours if using Humalog, Admelog/Insulin lispro Sanofi, or Lyumjev insulin; every 72 hours if using NovoLog/NovoRapid or Trurapi/Insulin aspart Sanofi insulin. Wash your hands with anti-bacterial soap before handling the infusion set and thoroughly clean the insertion site on your body to avoid infection. Contact your healthcare provider if you have symptoms of infection at your insulin infusion site.

⚠ PRECAUTION

CHANGE your cartridge every 48 hours if using Humalog, Admelog/Insulin lispro Sanofi, or Lyumjev insulin; every 72 hours if using NovoLog/NovoRapid or Trurapi/Insulin aspart Sanofi insulin. Wash your hands with anti-bacterial soap before handling the infusion set and thoroughly clean the insertion site on your body to avoid infection. Contact your healthcare provider if you have symptoms of infection at your insulin infusion site.

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© 2025 Tandem Diabetes Care, Inc. All rights reserved. Tandem Diabetes Care, Tandem logos, Control-IQ, and t:slim X2 are either registered trademarks or trademarks of Tandem Diabetes Care, Inc. in the United States and/or other countries. All third-party marks are the property of their respective owners. / Tous droits réservés. Tandem Diabetes Care, logos Tandem, Control-IQ, et t:slim X2 sont des marques déposées ou des marques commerciales de Tandem Diabetes Care, Inc. aux États-Unis et/ou dans d'autres pays. Toutes les autres marques de tierces parties appartiennent à leurs propriétaires respectifs.
■ Tandem Diabetes Care, Inc.