


Deliver an Extended Bolus

for the t:slim X2™ Insulin Pump




The instructions below are provided as a reference tool for caregivers who are already familiar with the use of an insulin pump and with insulin therapy in general. Not all screens are shown. For more detailed information on the operation of Tandem's insulin pumps, please visit www.tandemdiabetes.com/support.

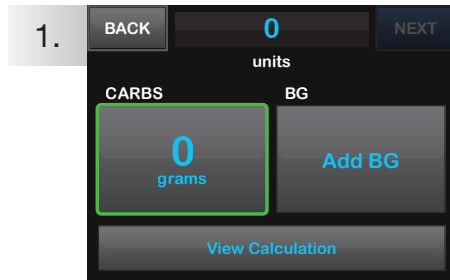


1. Tap **0 grams** to enter the carbs for your bolus.

NOTE: If this button reads "units," the carb feature is turned off in the active profile.

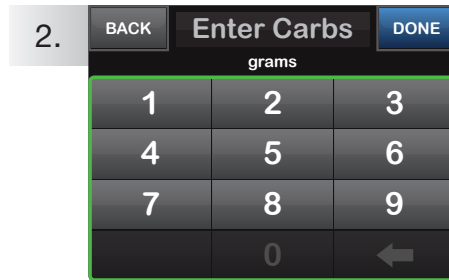


2. Enter desired value. Be sure "grams" is displayed above keypad for food boluses.

Tap **DONE** to continue.

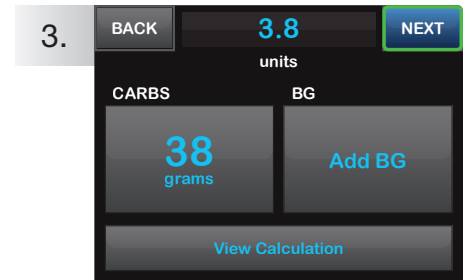


3. Tap **NEXT** to continue, then tap **YES** to confirm.

NOTE: If an above-target BG is entered, the correction bolus will not be extended.

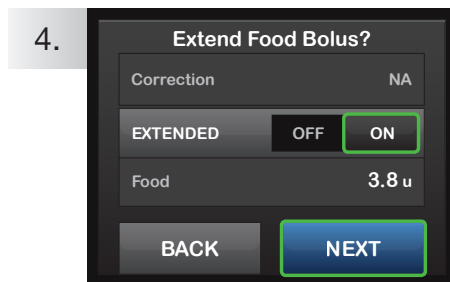


4. Tap **ON** to extend the bolus.


Tap **NEXT** to continue.

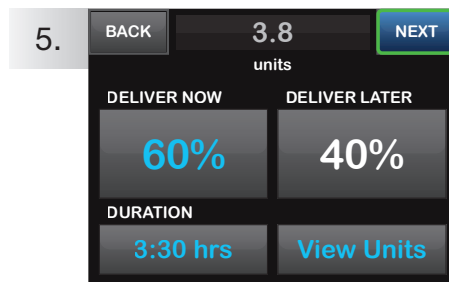

5. Tap **NEXT** to use default settings or tap the **DELIVER NOW** and **DURATION** buttons and set your desired values, then tap **NEXT** to continue.

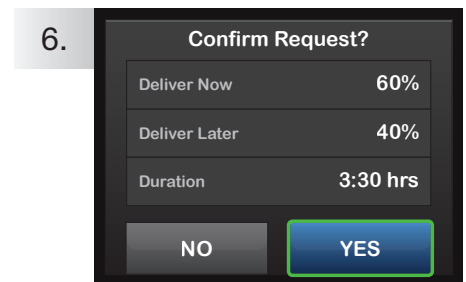


6. Tap **YES** to confirm.

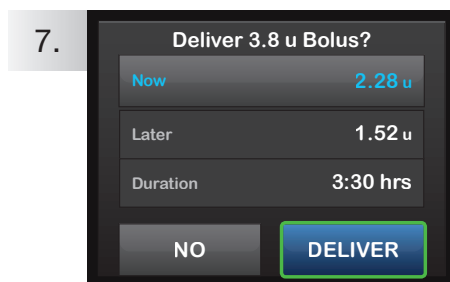

7. The delivery screen will confirm how much insulin will be delivered up front, how much will be delivered over time, and the delivery duration. Tap **DELIVER** to start the bolus.



8. The **BOLUS INITIATED** screen will appear to confirm delivery has started.

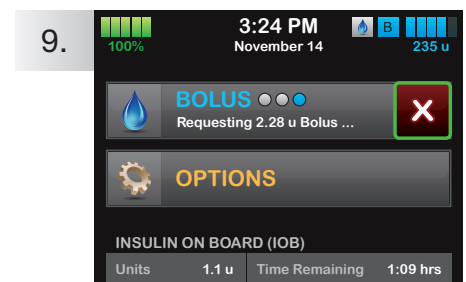

9. To cancel the undelivered portion of the bolus, tap the white X on the **BOLUS** button on the Home Screen, then tap **YES** to confirm canceled bolus.

