



Personal Profiles

on the t:slim X2 Insulin Pump


The instructions below are provided as a reference tool for caregivers who are already familiar with the use of an insulin pump and with insulin therapy in general. Not all screens are shown. For more detailed information on the operation of the Tandem t:slim X2™ insulin pump, please refer to its User Guide.

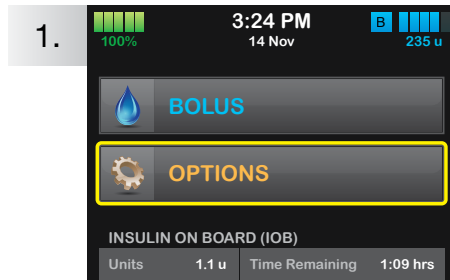


1. Tap **OPTIONS**.

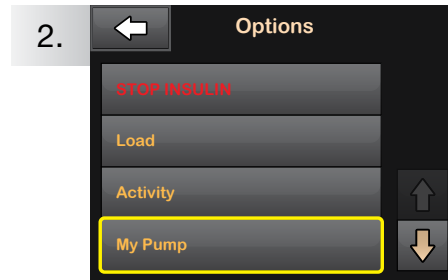


2. Tap **My Pump**.

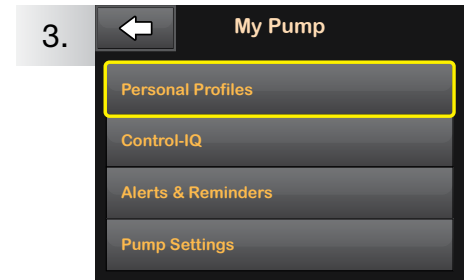


3. Tap **Personal Profiles**.

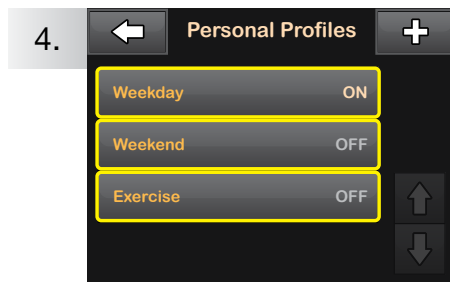


4. Tap the name of the Personal Profile to view or edit.

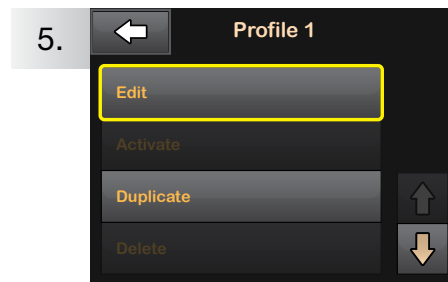

5. Tap **Edit** to edit or view the settings.

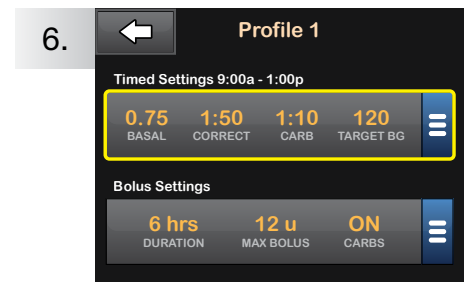

6. Tap your current settings to see the full list of timed settings for the entire day.

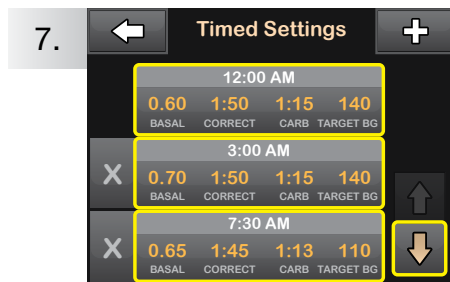

7. Tap the time segment the user wishes to edit.
If not all segments are visible, tap the **Down Arrow**.



8. Tap **Basal**, **Correction Factor**, **Carb Ratio**, or **Target BG** to make changes, then tap **Done**.
When finished, tap **Done**.



9. Confirm settings. Recent changes appear in orange.
Tap **Done** to confirm.



TANDEM
DIABETES CARE

(877) 801-6901
tandemdiabetes.com

t:simulator App
A free virtual pump demo

