





Tap **Options** from the Home screen and then **My Pump**




Tap **Personal Profiles**

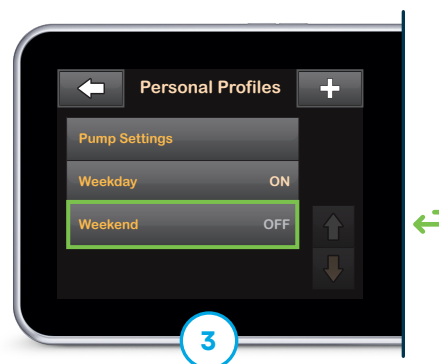

Tap the profile name that needs changes

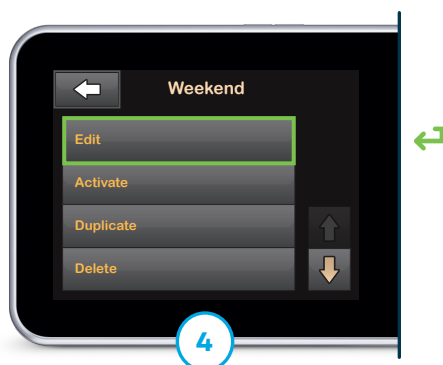

Tap **Edit** to make changes

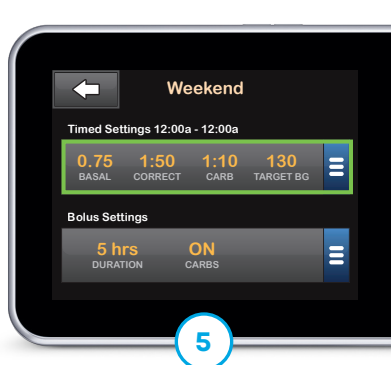

Tap **Timed Settings**




Tap the desired segment and then the setting that needs to be changed



Use the keypad to enter the new value. Tap ☒ to save




Tap ☒ to save the segment and then tap ☒ to confirm

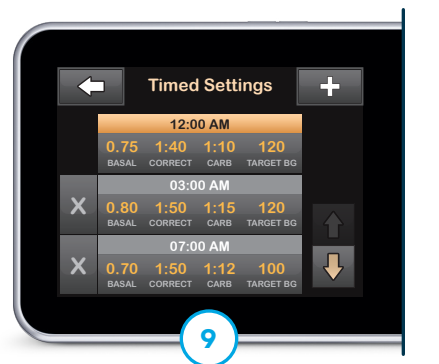

Tap other segments to make changes or return to the Home screen



A Temp Rate is used to increase or decrease the current Personal Profile's Basal Rate for a period of time.

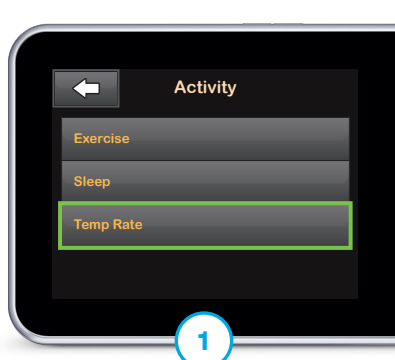

Tap **Options** from the Home screen, **Activity**, and then **Temp Rate**




Tap **Temp Rate**, enter the desired percentage, and then tap **✓** to continue



Tap **Duration**, enter the desired length of time, and then tap **✓** to continue. Tap **✓** again to confirm.



While active, a red or orange icon will display on the Home screen

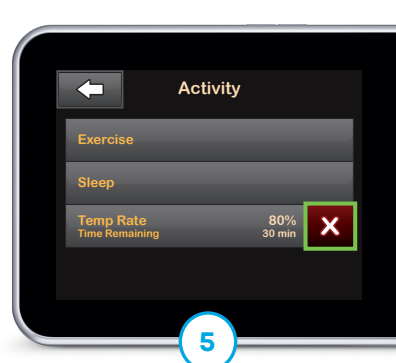

To stop at any time, tap **✗** on the **Activity** screen and then **✓** to confirm

Visit tandemdiabetes.com/support for additional resources.

These instructions are provided as a reference tool for individuals who are already familiar with the use of a Tandem Diabetes Care insulin pump. Not all screens are shown. For more detailed information on the operation of the t:slim X2 insulin pump, please refer to its user guide.

Important Safety Information: RX ONLY. Indicated for patients with type 1 diabetes, 2 years and older and for patients with type 2 diabetes, 18 years and older. Safety info: tandemdiabetes.com/safetyinfo.

WARNING: Control-IQ+ technology should not be used in anyone under the age of 2 years old with type 1 diabetes or under the age of 18 years old with type 2 diabetes. It should also not be used in patients who require less than a total daily insulin dose of 5 units of insulin per day or who weigh less than 20 pounds, as those are the required minimum values needed for Control-IQ+ to operate safely.

© 2025 Tandem Diabetes Care, Inc. All rights reserved. Tandem Diabetes Care, Tandem logos, t:slim X2, and Control-IQ+ are either registered trademarks or trademarks of Tandem Diabetes Care, Inc. in the United States and/or other countries. MAT-0199_1



TANDEM
Diabetes Care

(877) 801-6901
tandemdiabetes.com

FOR USE IN THE
UNITED STATES ONLY

