

# Stop and Resume Insulin Delivery

on Tandem Insulin Pumps




OPTIONS


The instructions below are provided as a reference tool for caregivers who are already familiar with the use of an insulin pump and with insulin therapy in general. Not all screens are shown. For more detailed information on the operation of Tandem's insulin pumps, please visit [www.tandemdiabetes.com/Support](http://www.tandemdiabetes.com/Support).




## Stop Insulin Delivery

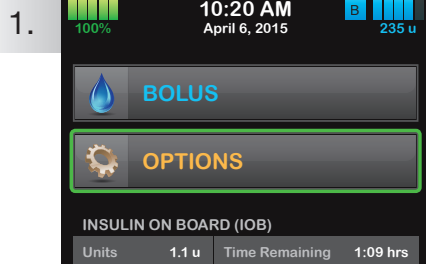

Tap **OPTIONS**.

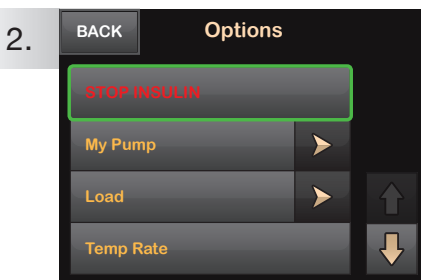


Tap **STOP INSULIN**.

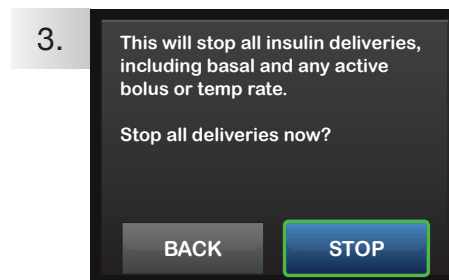


Tap **STOP**.




## Resume Insulin Delivery

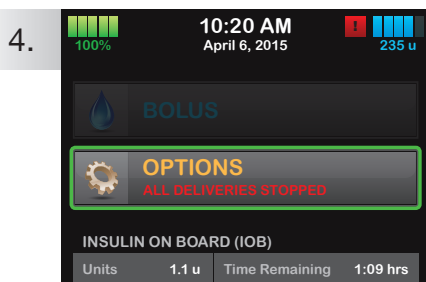

Tap **OPTIONS**.

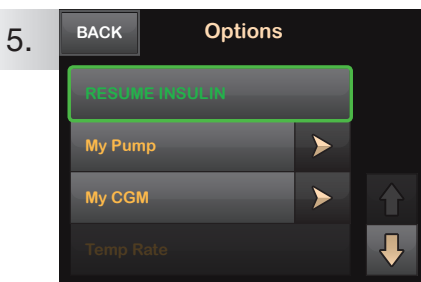

Tap **RESUME INSULIN**.

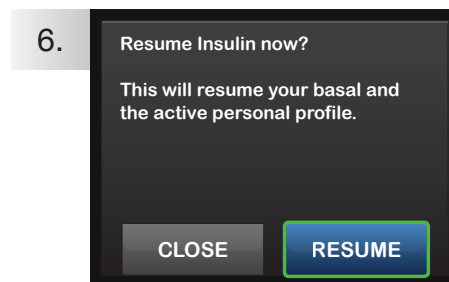

Confirm by tapping **RESUME**.

**NOTE:** Any temp rates or boluses active before you stopped delivery will not resume. It will resume your active personal profile.



(877) 801-6901  
[www.tandemdiabetes.com](http://www.tandemdiabetes.com)



TO DOWNLOAD APP:  
Text **DEMO** to  
**TANDEM (826336)**

