




The instructions below are provided as a reference tool for caregivers who are already familiar with the use of an insulin pump and with insulin therapy in general. Not all screens are shown. For more detailed information on the operation of the t:slim X2 insulin pump, please refer to its user guide.

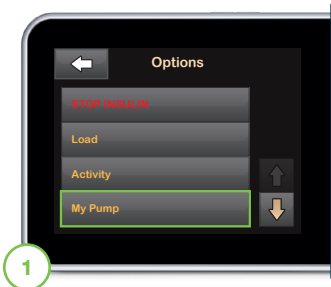

1 Tap **OPTIONS** from the Home screen and then **My Pump**.



2 Tap **Personal Profiles**.



3 Tap the Personal Profile name to view or edit.



4 Tap **Edit** to edit or view the settings.



5 Tap the current settings to see the full list of timed settings for the entire day.



6 Tap the time segment the user wishes to edit. If not all segments are visible, tap the **Down Arrow**.



7 Tap **Basal**, **Correction Factor**, **Carb Ratio**, or **Target BG** to make changes, then tap . When finished, tap .



8 Confirm settings. Recent changes appear in orange. Tap to confirm.

Important Safety Information: Tandem insulin pumps are intended for the subcutaneous delivery of insulin, at set and variable rates, for the management of diabetes mellitus in persons requiring insulin. The t:slim X2 insulin pump is approved for individuals 6 years of age and older. For Important Safety Information, please visit tandemdiabetes.com/safetyinfo.

© 2022 Tandem Diabetes Care, Inc. All rights reserved. Tandem Diabetes Care and t:slim X2 are either registered trademarks or trademarks of Tandem Diabetes Care, Inc. in the United States and/or other countries. All third-party marks are the property of their respective owners. ML-1004685_B

