



The instructions below are provided as a reference tool for caregivers who are already familiar with the use of an insulin pump and with insulin therapy in general. Not all screens are shown. For more detailed information on the operation of the t:slim X2 insulin pump, please refer to its user guide.



1

Tap **0 grams** to enter the carbs for the bolus.

Note: If this area reads "units," the carb feature is turned off in the active profile.



2

Enter desired value. Be sure "grams" is displayed above keypad for food boluses. Tap to continue.



3

Tap to continue, then tap to confirm.

Note: If an above-target blood glucose (BG) is entered, the correction bolus will not be extended.

Note: If certain advanced features are being used, the current Dexcom G6 CGM reading may auto-populate to the bolus calculator. Please see the User Guide for more information.



4

Tap the toggler to the ON position to extend the bolus. Tap to continue.



5

Tap to use default settings or tap **DELIVER NOW** and **DURATION** and set the desired values, then tap to continue.



6

Tap to confirm.

Note: If Control-IQ technology is turned on, extended boluses are limited to two hours. Please see the User Guide for more information.



7

The delivery screen will confirm how much insulin will be delivered up front, how much will be delivered over time, and the delivery duration. Tap to start the bolus. The BOLUS INITIATED screen will appear to confirm delivery has started.



8

To cancel the undelivered portion of the bolus, tap "X" next to BOLUS on the Home screen, then tap to confirm canceled bolus.

Important Safety Information: Tandem insulin pumps are intended for the subcutaneous delivery of insulin, at set and variable rates, for the management of diabetes mellitus in persons requiring insulin. The t:slim X2 insulin pump is approved for individuals 6 years of age and older. For Important Safety Information, please visit tandemdiabetes.com/safetyinfo.

© 2022 Tandem Diabetes Care, Inc. All rights reserved. Tandem Diabetes Care, Control-IQ, and t:slim X2 are either registered trademarks or trademarks of Tandem Diabetes Care, Inc. in the United States and/or other countries. Dexcom and Dexcom G6 are registered trademarks of Dexcom, Inc. in the United States and/or other countries. ML-1004691_B

