


Set a Temp Basal Rate


for the t:slim X2™ Insulin Pump




The instructions below are provided as a reference tool for caregivers who are already familiar with the use of an insulin pump and with insulin therapy in general. Not all screens are shown. For more detailed information on the operation of Tandem's insulin pumps, please visit www.tandemdiabetes.com/support.

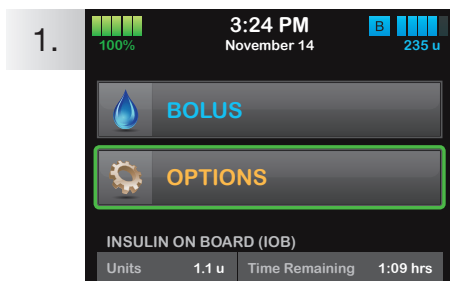


1. Tap **OPTIONS**.

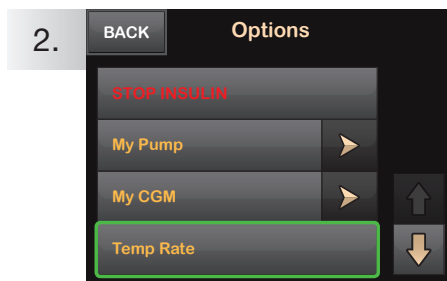

2. Tap **Temp Rate**.

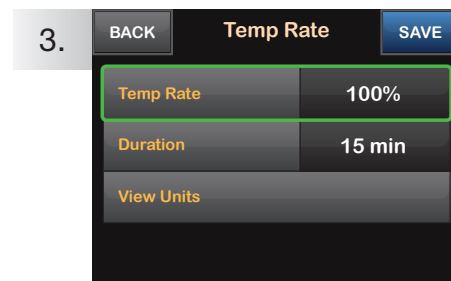


3. Tap **Temp Rate**.

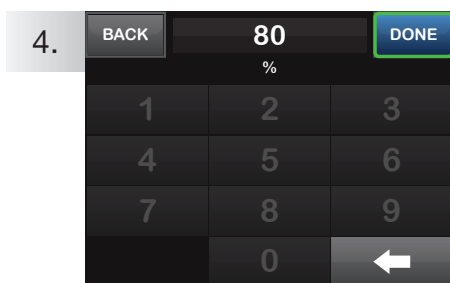


4. Using the onscreen keypad enter desired percentage. Tap **DONE**.

NOTE: Current rate is 100%. An increase is greater than 100% and a decrease is less than 100%.

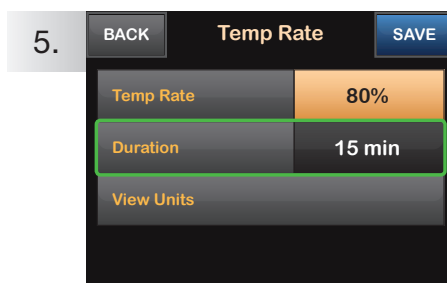

5. Tap **Duration**. Using the onscreen keypad enter desired length of time for Temp Rate. Tap **DONE**.


NOTE: To see the actual units to be delivered, tap **View Units**.

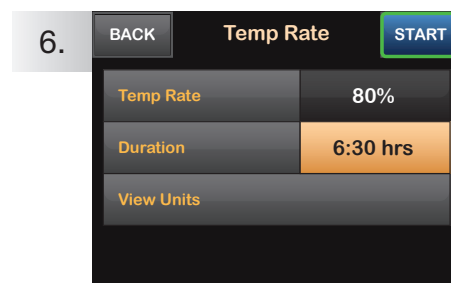

6. Verify settings and tap **START**.




7. The **TEMP RATE STARTED** screen will appear to confirm the Temp Rate has started.

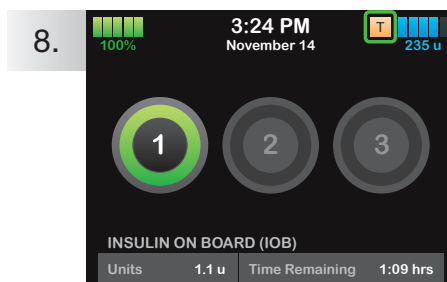

8. The **Screen Lock** screen will appear with the orange **T** icon indicating a Temp Rate is active.

NOTE: If a Temp Rate of 0% is currently active, the orange **T** icon will be replaced with a red **T** icon.

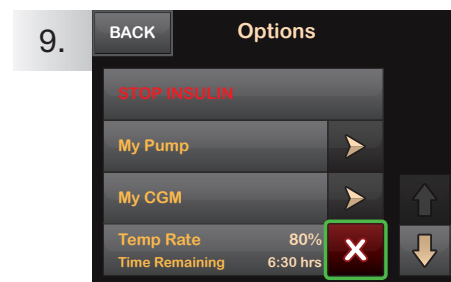

9. **NOTE:** To stop Temp Rate at any time, tap **OPTIONS**, then tap the white **X**. A confirmation screen will appear. Tap **STOP**.



TANDEM
DIABETES CARE

(877) 801-6901
www.tandemdiabetes.com

t:simulator App

A demo as simple as our pumps!



DOWNLOAD IT TODAY:
Text **DEMO** to
TANDEM (826336)

