

t:case™ and t:case 480

INSTRUCTIONS FOR USE

The t:case and t:case 480 are designed to allow charging and cartridge changes without removing the pump from the case and can be used with or without the supplied metal clip. On the t:case, the clip can be oriented both vertically or horizontally, while the t:case 480 must be clipped vertically because of the larger cartridge.

To place the pump in the case:

1. Disconnect from your site.
2. Line up the wake button of the pump with the button on the top of the case, and thread the tubing through the cutout (Figure A).
3. Once the tubing is securely through the cutout, insert the right side of the pump into the case, then press down on the left side.

** If your pump and case buttons are not on the same side, then your pump is in the case upside down and will not properly sit within the case.*

To remove the pump from the case:

1. Disconnect from your site.
2. Place the pump face down (away from you).
3. Press down with your thumbs while simultaneously pulling back on the thin case rim with your index fingers (Figure B).

To change the orientation of the clip (t:case only):

1. Remove the pump from the case.
2. Once the pump is removed, using your fingernail or a small flathead screwdriver, lightly lift up the plastic tab behind the clip and slide the clip out (Figure C).
3. Slide the clip into the other slot until it clicks.



To watch a video on the t:case and t:case 480, please visit
www.tandemdiabetes.com/training.



CUSTOMER TECHNICAL SUPPORT
AVAILABLE 24/7 | (877) 801-6901