




Tap **BOLUS** from the Navigation bar.

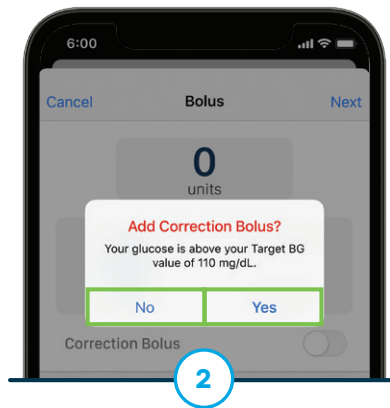


If the blood glucose (BG) value is above target, tap to add or decline a Correction Bolus.

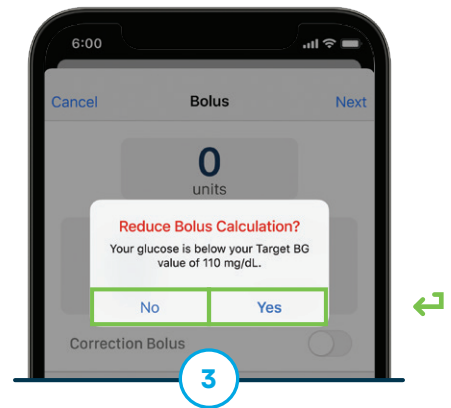



If it's below target, but above 70 mg/dL, tap to reduce the amount or decline change.

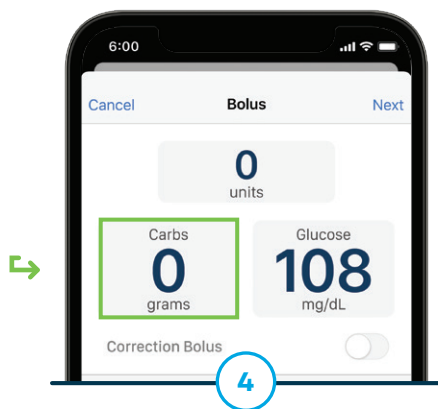


Tap **0 grams** to enter the carbohydrate amount.

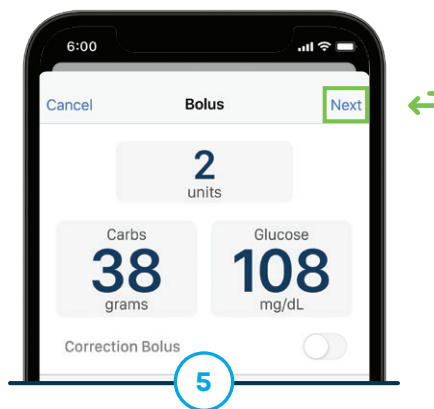


Tap **Next** to save values.

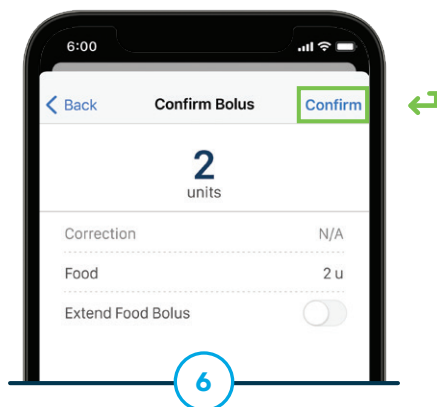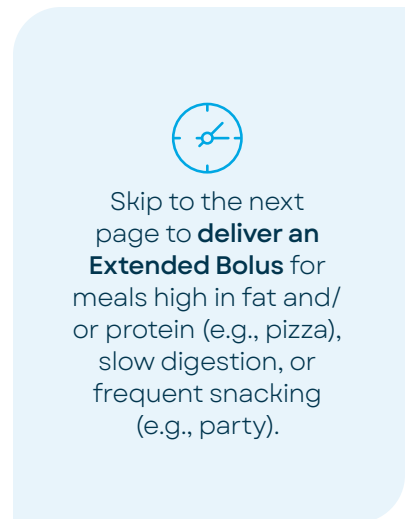


Tap **Confirm** to verify request.

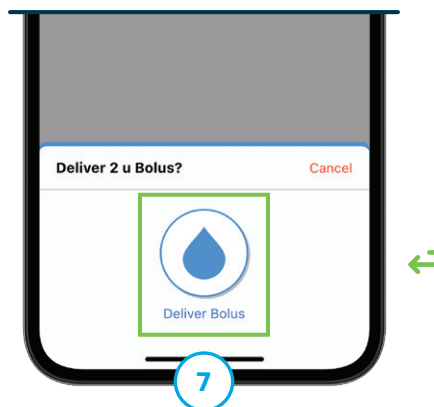


Tap droplet to deliver bolus.

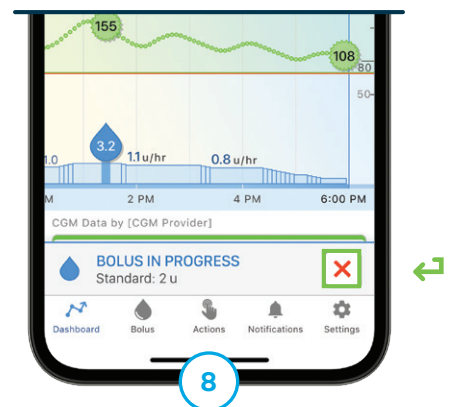


To cancel bolus, tap **X** on the Dashboard and then tap **Yes** to confirm.

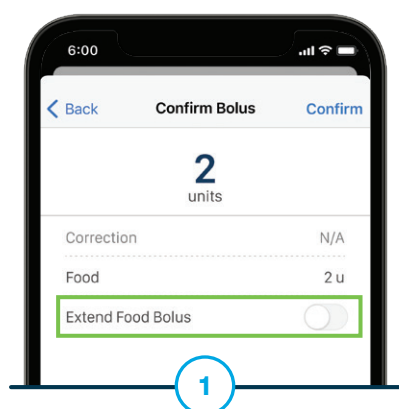


Tap the toggle on.

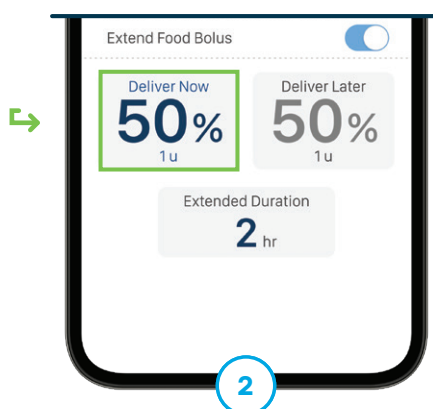


Use the default settings or tap the DELIVER NOW percentage to update the amount.

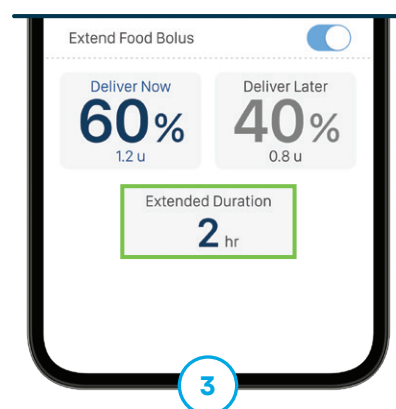


If desired, tap **2 hrs** to change the length of time.

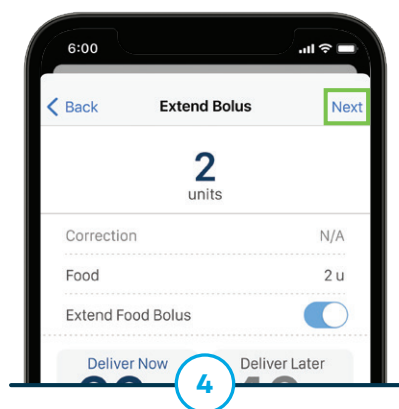

Tap **Next** to save values.

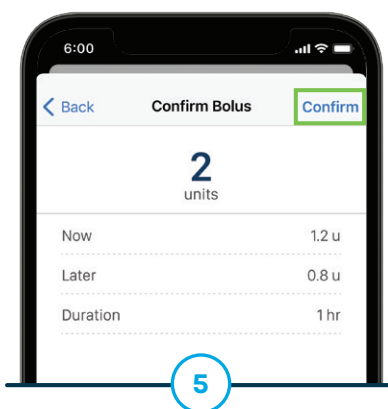

Tap **Confirm** to verify request.

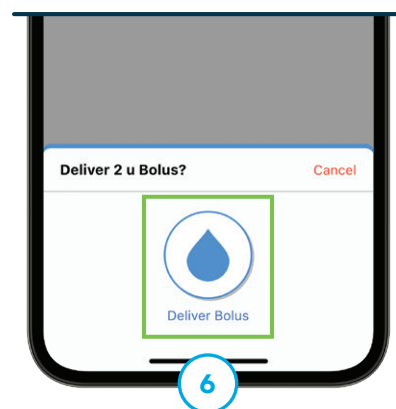

Tap droplet to deliver bolus.

Visit tandemdiabetes.com/support for additional resources.

These instructions are provided as a reference tool for individuals who are already familiar with the use of a Tandem Diabetes Care insulin pump. Not all screens are shown. For more detailed information on the operation of the Tandem Mobi system, please refer to its user guide.

Important Safety Information: RX ONLY. The Tandem Mobi insulin pump with Control-IQ+ technology is intended for the management of type 1 diabetes mellitus in individuals 2 years of age and greater. Visit tandemdiabetes.com/safetyinfo for additional important safety information.

WARNING: Control-IQ+ technology should not be used by anyone under the age of 2 years old, who require less than 5 units of insulin per day, or who weigh less than 20 pounds.

© 2025 Tandem Diabetes Care, Inc. All rights reserved. Tandem Diabetes Care, Tandem logos, Tandem Mobi, and Control-IQ+ are either registered trademarks or trademarks of Tandem Diabetes Care, Inc. in the United States and/or other countries. ML-1012732_B



TANDEM
Diabetes Care

(877) 801-6901
tandemdiabetes.com

FOR USE IN THE
UNITED STATES ONLY

