



Tap **BOLUS** from the Home screen.



If the blood glucose (BG) value is above target, tap ☒ to add a Correction Bolus or ☐ to decline.

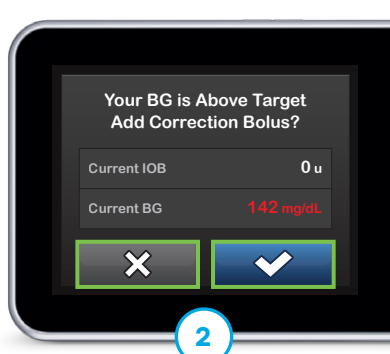


If it's below target, but above 70 mg/dL, tap ☒ to reduce the amount or ☐ to decline.

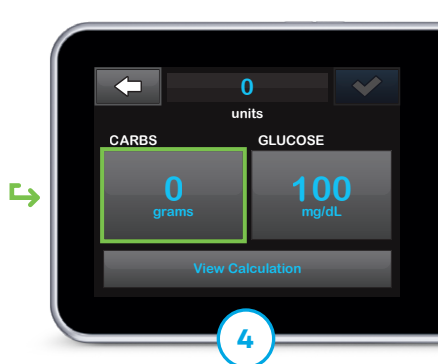

Tap **0 grams** to enter the carbohydrate amount.




Tap ☒ to save values.

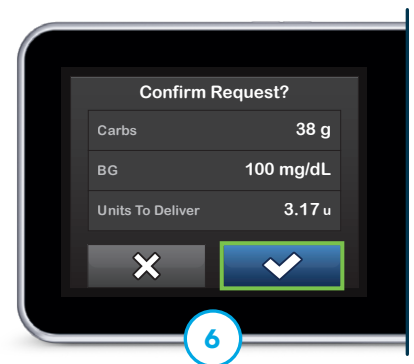

Tap ☒ to confirm request.



Skip to the next page to **deliver an Extended Bolus** for meals high in fat and/or protein (e.g., pizza), slow digestion, or frequent snacking (e.g., party).

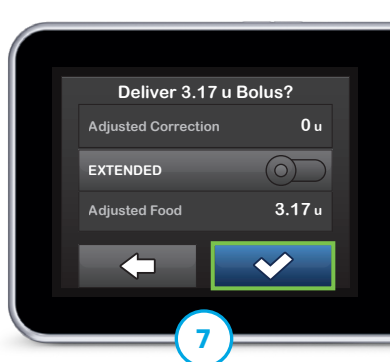

Tap ☒ to deliver bolus.



To cancel bolus, tap ☒ on the Home screen and then ☒ to confirm.



Tap the toggle on.




Tap to continue.



Use the default settings or tap the DELIVER NOW percentage to update the amount.



If desired, tap **2 hrs** to change the length of time.

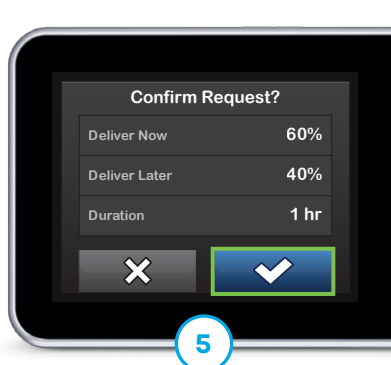

Tap to save values and then tap again to confirm request.



Tap to deliver the Extended Bolus.

Visit [tandemdiabetes.com/support](https://tandemdiabetes.com/support) for additional resources.

These instructions are provided as a reference tool for individuals who are already familiar with the use of a Tandem Diabetes Care insulin pump. Not all screens are shown. For more detailed information on the operation of the t:slim X2 insulin pump, please refer to its user guide.

**Important Safety Information:** RX ONLY. The t:slim X2 insulin pump with Control-IQ+ technology is intended for the management of type 1 diabetes mellitus in individuals 2 years of age and greater. Visit [tandemdiabetes.com/safetyinfo](https://tandemdiabetes.com/safetyinfo) for additional important safety information.

**WARNING:** Control-IQ+ technology should not be used by anyone under the age of 2 years old, who require less than 5 units of insulin per day, or who weigh less than 20 pounds.

© 2025 Tandem Diabetes Care, Inc. All rights reserved. Tandem Diabetes Care, Tandem logos, t:slim X2, and Control-IQ+ are either registered trademarks or trademarks of Tandem Diabetes Care, Inc. in the United States and/or other countries. ML-000312\_J



**TANDEM**  
Diabetes Care

(877) 801-6901  
[tandemdiabetes.com](https://tandemdiabetes.com)

FOR USE IN THE  
UNITED STATES ONLY

