




Tap **Settings** from the Navigation bar and then **Pump**.

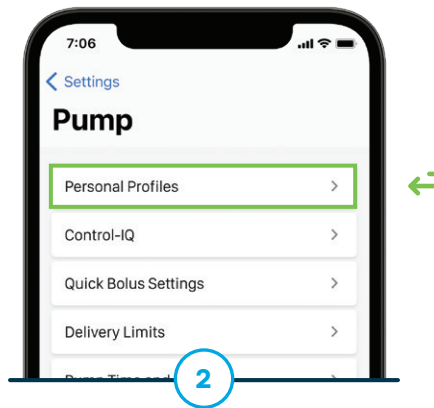

Tap **Personal Profiles**.

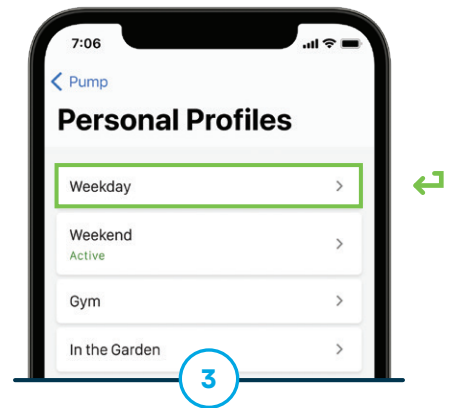


Tap the profile name that needs changes.

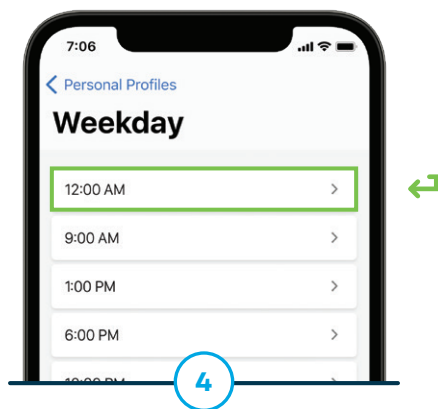


Tap the desired segment to make changes.




Tap the setting that needs to be changed.

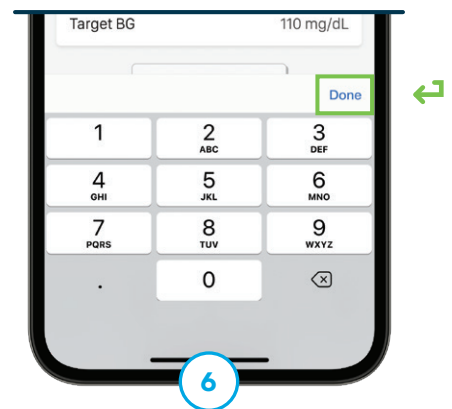

Use the keypad to enter the new value and then tap **Done**.

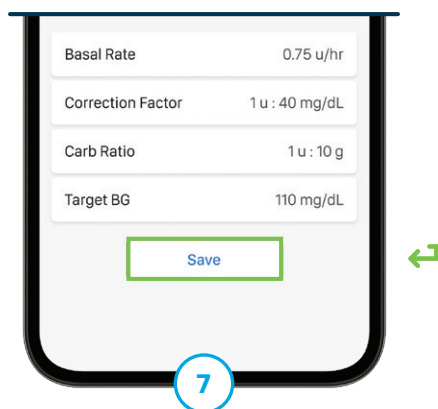

Tap **Save** to exit the segment.

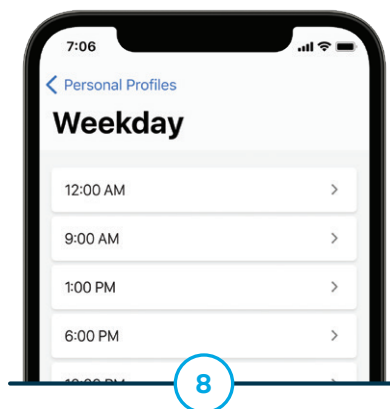


Tap other segments to make changes or return to the Dashboard.



The Personal Profile can be activated, duplicated, deleted, and/or renamed from the Profile Options menu.



A Temp Rate is used to increase or decrease the current Personal Profile's Basal Rate for a period of time.

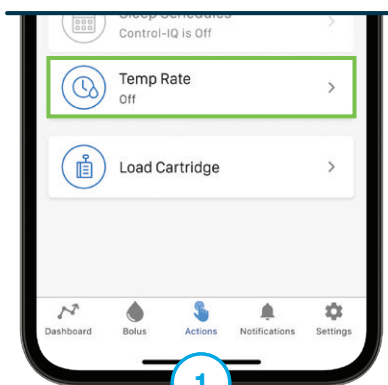


Tap **Actions** from the Navigation bar and then **Temp Rate**.




Tap **Temp Rate** and enter the desired percentage.



Tap **Duration** and enter the desired length of time. Tap **Start** to begin.

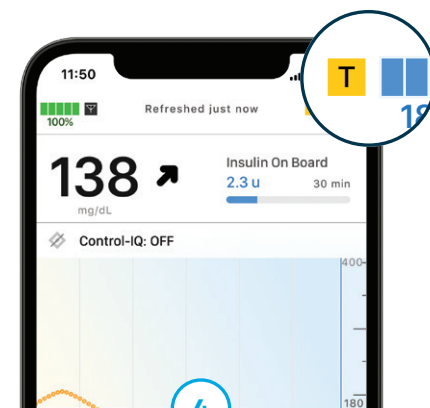

While active, a red or orange icon will display on the Dashboard.

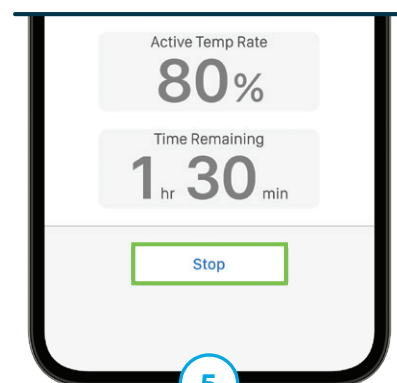

To cancel at any time, tap **Stop** on the **Temp Rate** screen and then **Yes** to confirm.

Visit tandemdiabetes.com/support for additional resources.

These instructions are provided as a reference tool for individuals who are already familiar with the use of a Tandem Diabetes Care insulin pump. Not all screens are shown. For more detailed information on the operation of the Tandem Mobi system, please refer to its user guide.

Important Safety Information: RX ONLY. The Tandem Mobi insulin pump with Control-IQ+ technology is intended for the management of type 1 diabetes mellitus in individuals 2 years of age and greater. Visit tandemdiabetes.com/safetyinfo for additional important safety information.

WARNING: Control-IQ+ technology should not be used by anyone under the age of 2 years old, who require less than 5 units of insulin per day, or who weigh less than 20 pounds.

© 2025 Tandem Diabetes Care, Inc. All rights reserved. Tandem Diabetes Care, Tandem logos, Tandem Mobi, and Control-IQ+ are either registered trademarks or trademarks of Tandem Diabetes Care, Inc. in the United States and/or other countries. ML-1012936_B



TANDEM
Diabetes Care

(877) 801-6901
tandemdiabetes.com

FOR USE IN THE
UNITED STATES ONLY

