






Insert a new **FreeStyle Libre 3 Plus** sensor and pair the Tandem t:slim mobile app with the t:slim X2 insulin pump before starting these steps.

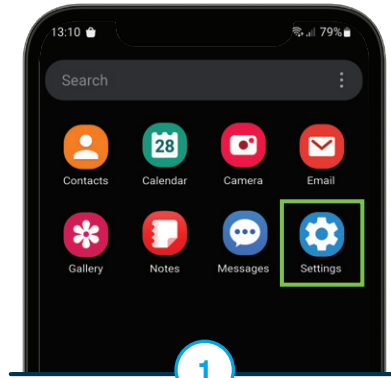


For iOS users, skip to step 4. For Android users, tap **Settings** from the smartphone's Home screen.

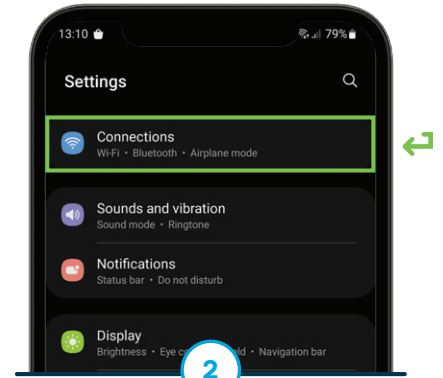


Tap **Connections** to continue.

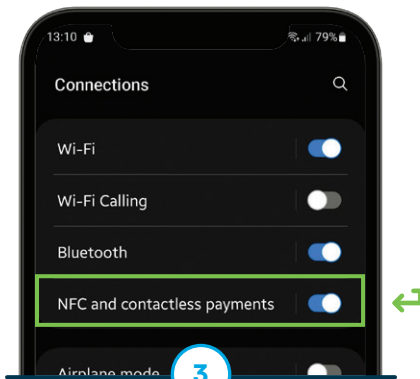


Make sure the NFC and contactless payments option is enabled.

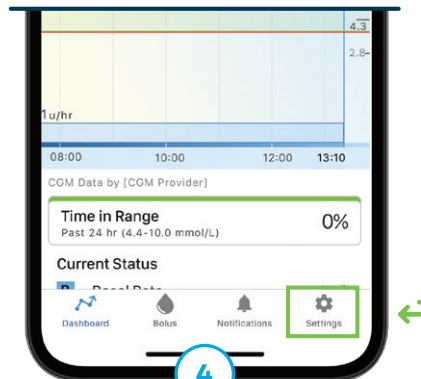


Tap **Settings** from the Navigation bar of the Tandem t:slim mobile app.

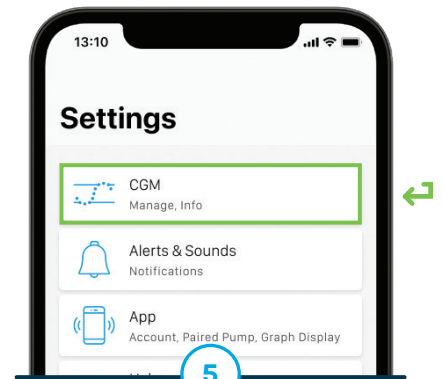


Tap **CGM** and then **FreeStyle Libre 3 Plus** to continue.

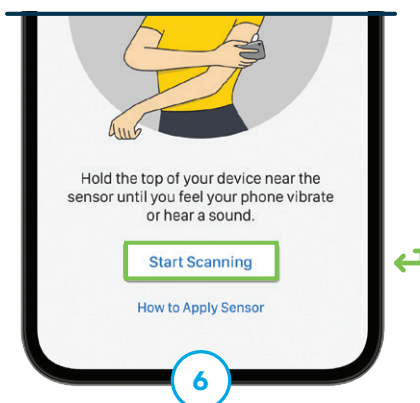

For Android users, skip to next step. For iOS users, tap **Start Scanning** to continue.

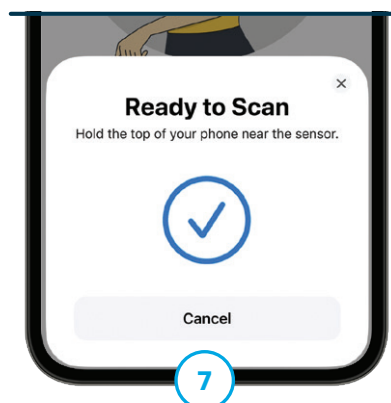


Hold the top (iOS) or back (Android) of the smartphone near the sensor until it vibrates or an alert is heard.

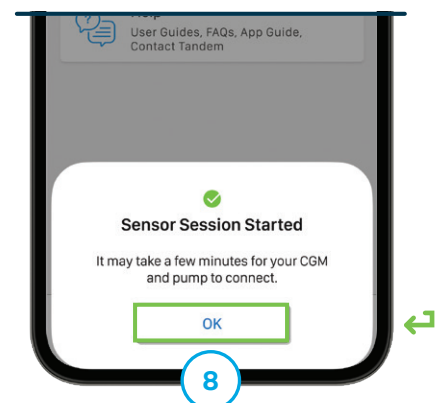

The Sensor Session Started screen is displayed. Tap **OK**.

Start Sensor Session (continued)

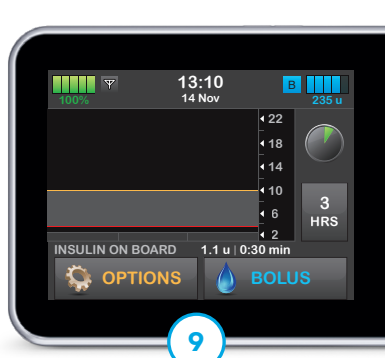

A CGM trend graph and 1-hour sensor startup countdown symbol will appear.



The symbol fills in over time to show how much time is left.



Once startup is complete, the new session will begin.

Visit tandemdiabetes.com/support for additional resources.

These instructions are provided as a reference tool for individuals who are already familiar with the use of a Tandem Diabetes Care insulin pump. Not all screens are shown. For more detailed information on the operation of the t:slim X2 pump, please refer to its user guide.

Important Safety Information: Tandem insulin pumps are intended for the subcutaneous delivery of insulin, at set and variable rates, for the management of diabetes mellitus in persons requiring insulin. Visit tandemdiabetes.com/safetyinfo for additional important safety information.

WARNING: Control-IQ+ technology should not be used in anyone under the age of 2 years old with type 1 diabetes. It should also not be used in patients who require less than a total daily insulin dose of 5 units of insulin per day or who weigh less than 20 pounds (9 kilograms), as those are the required minimum values needed for Control-IQ+ to operate safely.

© 2025 Tandem Diabetes Care, Inc. All rights reserved. Tandem Diabetes Care, Tandem logos, t:slim, t:slim X2, and Control-IQ+ are either registered trademarks or trademarks of Tandem Diabetes Care, Inc. in the United States and/or other countries. The sensor housing, FreeStyle, Libre, and related brand marks are marks of Abbott and used with permission. iOS is a trademark or registered trademark of Cisco in the U.S. and other countries and is used under license. Android is a trademark of Google LLC. MAT-0299_1