

View Status (with CGM)



The instructions below are provided as a reference tool for caregivers who are already familiar with the use of an insulin pump and with insulin therapy in general. Not all screens are shown. For more detailed information on the operation of the t:slim X2 insulin pump, please refer to its user guide.



Tap the insulin level icon in the upper right corner of the Home screen.



Your pump will display the name of the user's active profile, the current basal rate, the time and amount of the last bolus, and whether or not they are using Basal-IQ technology.

Note: If the user has an active Extended Bolus, this screen will show how much insulin has been delivered out of the total amount requested.



Tap the **Down Arrow** to display the current Correction Factor, Carb Ratio, Target BG, and Insulin Duration.



Tap the **Down Arrow** to display the Last Calibration, Time Sensor Started, Transmitter Battery status, and Mobile Connection status.

Explanation of icons

The following icons may appear in the areas to the left or right of the time and date on the Home screen of the t:slim X2 insulin pump.



Icon	Explanation
1	A system reminder, alert, error, or alarm is active.
	A bolus is being delivered.
В	Basal insulin is programmed and being delivered.
Т	A temporary basal rate is active.
Т	A temporary basal rate of 0 u/hr is active.

Icon	Explanation
0	A basal rate of 0 u/hr is active.
S	The Basal-IQ technology is currently active. All insulin deliveries have been suspended.
Y	CGM sensor session is active, and the transmitter is communicating with the pump.
Y	CGM sensor session is active, but the transmitter is not communicating with the pump.



View Status (without CGM)





The instructions below are provided as a reference tool for caregivers who are already familiar with the use of an insulin pump and with insulin therapy in general. Not all screens are shown. For more detailed information on the operation of the t:slim X2 insulin pump, please refer to its user guide.



Tap the insulin level icon in the upper right corner of the Home screen.



Your pump will display the name of the user's active profile, the current basal rate, the time and amount of the last bolus, and whether or not they are using Basal-IQ technology.

Note: If the user has an active Extended Bolus, this screen will show how much insulin has been delivered out of the total amount requested.



Tap the **Down Arrow** to display the current Correction Factor, Carb Ratio, Target BG, and Insulin Duration.



Tap the **Down Arrow** to display the Last Calibration, Time Sensor Started, Transmitter Battery status, and Mobile Connection status.

Explanation of icons

The following icons may appear in the areas to the left or right of the time and date on the Home screen of the t:slim X2 insulin pump.



Icon	Explanation
1	A system reminder, alert, error, or alarm is active.
	A bolus is being delivered.
В	Basal insulin is programmed and being delivered.
T	A temporary basal rate is active.
T	A temporary basal rate of 0 u/hr is active.

Icon	Explanation
0	A basal rate of 0 u/hr is active.
1	All insulin deliveries are stopped.

Important Safety Information: RX ONLY. Tandem insulin pumps are intended for the subcutaneous delivery of insulin, at set and variable rates, for the management of diabetes mellitus in persons requiring insulin. The t:slim X2 insulin pump is approved for individuals 6 years of age and older. For Important Safety Information, please visit tandemdiabetes.com/safetyinfo.

© 2022 Tandem Diabetes Care, Inc. All rights reserved. Tandem Diabetes Care, Basal-IQ, and t:slim X2 are either registered trademarks or trademarks of Tandem Diabetes Care, Inc. in the United States and/or other countries. ML-000316_F

