



The Tandem Mobi system is so small that it can be worn almost anywhere, giving users greater discretion and comfort. This guide will provide some tips on how to wear and take care of the system.



Wear it on-body with a lightweight adhesive sleeve\* or clipped to clothing.




Slip it into the coin pocket of jeans (yes, it's that small!)

## Freedom to Disconnect

The Tandem Mobi pump can easily be disconnected from its infusion set site for a number of activities, including physical sports, saunas, swimming, and other spontaneous activities.

Different insertion angles and a variety of tubing length options provide even more ways the pump can be worn.

**Flip to back** for full list of options.

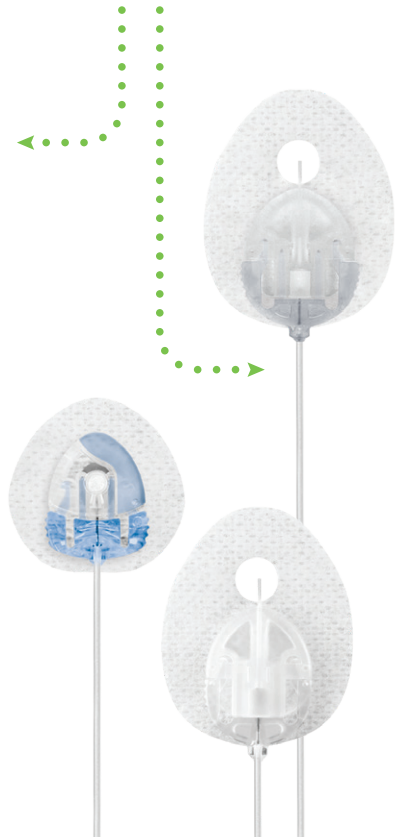



## Things to Consider

---

- ✓ Orient the adhesive sleeve so that the LED Status Indicators on the insulin pump can be seen
- ✓ Consider the placement of the infusion set prior to placing the adhesive sleeve to avoid stretching the tubing
- ✓ Do not wear or place the pump more than 12 inches above the infusion site. It is also recommended that the pump and smartphone be worn at least 6.5 inches apart.
- ✓ When integrated with continuous glucose monitoring (CGM), wear the pump on the same side of the body as the CGM sensor
- ✓ Avoid exposing the pump to magnets or products with magnets
- ✓ Be careful when applying sunscreens, bug sprays, and lotions. Some products can cause the pump and cartridge plastic to crack which may result in over- or under-delivery of insulin.†

† If the Tandem Mobi system becomes exposed to household chemicals or skin care products, be sure to clean the pump immediately. Always wash your hands before handling the pump or cartridge after applying skin care products like sunscreen and bug spray. If you need to apply any of these products to your skin, we recommend you temporarily disconnect the infusion site from your body while applying and wash your hands thoroughly before connecting your infusion site back to your body. It's also important to avoid excessive exposure of your filled cartridge to direct sunlight as this may impact the effectiveness of insulin.

## How to Clean the Pump

It is recommended that the Tandem Mobi insulin pump be:

- ✓ Routinely cleaned, especially if exposed to dirt, dust, and skin care products
- ✓ Use a lint-free cloth dampened with warm water and a bit of mild dish detergent to wipe the pump
- ✓ Never submerge it in water or use any other liquid to clean it
- ✓ Use a soft towel to dry the pump

▀ **Note:** Before cleaning, always suspend your insulin, disconnect your infusion set, and remove the cartridge.



### Users can have peace of mind in the event of accidental submersion in water.

The Tandem Mobi insulin pump is water resistant to a depth of 8 feet for up to 2 hours (IP28 rating). We do not recommend swimming, scuba diving, surfing, or any other activities that could submerge the pump in water for an extended period of time.

## Available Infusion Sets

Finding the right infusion set is an important part of successful insulin pump therapy. Choose from a variety of options.

Options	Tubing	Angles	Cannula
AutoSoft XC	5, 23, 32 or 43 in.	90°	6 or 9 mm
AutoSoft 30	23 or 43 in.	30°	13 mm
AutoSoft 90	23 or 43 in.	90°	6 or 9 mm
TruSteel	23 or 32 in.	90°	6 or 8 mm
VariSoft	23, 32, or 43 in.	20°-45°	13 or 17 mm

\* The Tandem Mobi adhesive sleeve is sold separately.

**Important Safety Information:** RX ONLY. **Indications for Use:** Tandem Mobi insulin pump with interoperable technology: The Tandem Mobi insulin pump with interoperable technology (the pump) is intended for the subcutaneous delivery of insulin, at set and variable rates, for the management of diabetes mellitus in persons requiring insulin. The pump is able to reliably and securely communicate with compatible, digitally connected devices, including automated insulin dosing software, to receive, execute, and confirm commands from these devices. The pump is intended for single patient, home use and requires a prescription. The pump is indicated for use in individuals 6 years of age and greater.

**Warning:** Do not use the Tandem Mobi insulin pump before consulting with your healthcare provider. Incorrect settings can result in over-delivery of insulin (which could lead to hypoglycemia) or under-delivery of insulin (which could lead to hyperglycemia). Visit [tandemdiabetes.com/safetyinfo](https://tandemdiabetes.com/safetyinfo) for additional important safety information.

These [infusion sets](#) are indicated for the subcutaneous infusion of insulin for the treatment of diabetes. These infusion sets are indicated for single use. For contraindications, warnings, precautions, and other important information, please refer to the instructions for use accompanying the infusion sets.

© 2023 Tandem Diabetes Care, Inc. All rights reserved. Tandem Diabetes Care, the Tandem logo, AutoSoft, VariSoft, TruSteel, and Tandem Mobi are either registered trademarks or trademarks of Tandem Diabetes Care, Inc. in the United States and/or other countries. Dexcom, Dexcom G6, and any related logos and design marks are either registered trademarks or trademarks of Dexcom, Inc. in the United States and/or other countries. ML-1012730\_A

